


## Directions for Use (Single Patient Use Only)

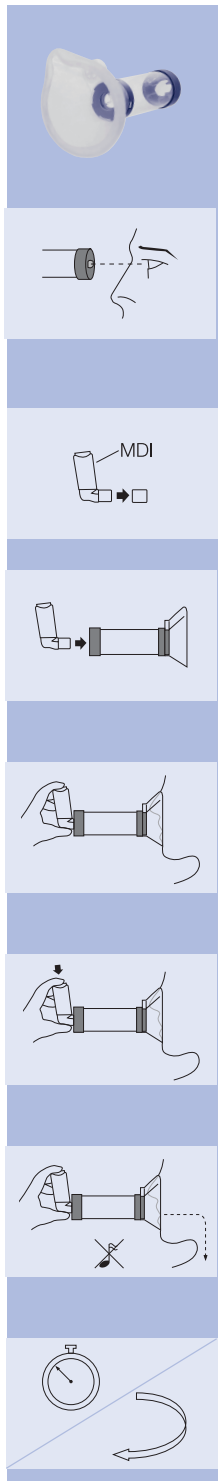

### ENGLISH

Carefully examine the product for damage, missing parts or foreign objects. Any foreign objects should be removed. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.

Before use, make sure that instructions supplied with inhaler (MDI) have been read.  
Remove cap.


Shake the inhaler (MDI) well immediately before each use. Insert the inhaler (MDI) into the back piece of the chamber.

Apply mask to face and ensure that there is a good seal.






Breathe out gently and depress inhaler (MDI) at beginning of slow inhalation. Maintain seal for 2-3 breaths after depressing inhaler (MDI). Administer one (1) puff at a time..

Slow down inhalation if you hear the **FLOWSignal®** Whistle sound.

Follow instructions supplied with the inhaler (MDI) on amount of time to wait before repeating steps 3-6 as prescribed.

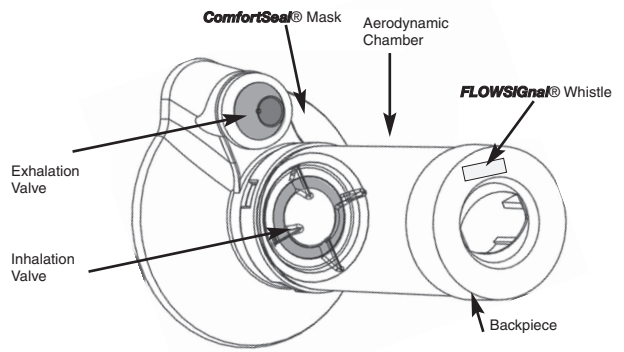

### Cleaning Instructions

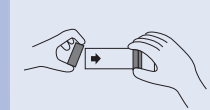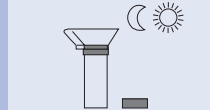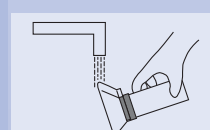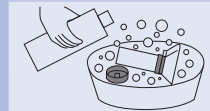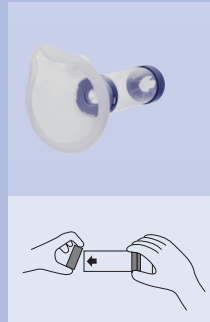

### ENGLISH

Clean **AeroChamber Plus® VHC** only as per instructions before first use, then weekly.

Remove back piece only. Do not remove mask or tamper with valves during cleaning.

Soak both parts for 15 minutes in luke warm water with liquid detergent. Agitate gently.

Rinse facemask only in water.

Shake out excess water.  
**Do not rub dry.**

Let air dry in vertical position.

Replace back piece when unit is completely dry and ready for use.

# AeroChamber Plus®

Valved Holding Chamber with **FLOWSignal®** Whistle with **ComfortSeal®** Mask

Distributed by:

**Forest Pharmaceuticals, Inc.**

Subsidiary of Forest Laboratories, Inc. St. Louis, MO 63045  
Monaghan Medical Corporation, 5 Latour Ave., Suite 1600, Plattsburgh, NY 12901  
© 1999,2005 Monaghan Medical Corporation

Covered by one or more of the following Patents 5,042,467; 5,645,049; 5,848,588; 5,988,160; 6,293,279; 6,345,617; 6,435,177; and patents pending  
Printed in USA RMC 5263 Revision: 08/08  
© 2001,2005 Forest Laboratories, Inc.

**AeroChamber Plus** and **ComfortSeal** are registered trademarks used under license.



### Cautions

- Read all instructions before use.
- Do not leave **AeroChamber Plus® VHC** unattended with children. This is a medical device, not a toy.
- Do not disassemble the product beyond what is recommended in the Cleaning Instructions or damage may result.
- If unsure on how to use this product, talk to your physician or pharmacist.

### Notes

- VHC = Valved Holding Chamber
- This product contains no latex.
- The **AeroChamber Plus® VHC** may need to be replaced after 12 months of continuous use. Environmental conditions, storage and proper cleaning may affect life span.

**Indications for Use:** The **AeroChamber Plus®** Valved Holding Chamber ("VHC") is intended to be used by patients who are under the care or treatment of a licensed health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers, prescribed by a physician or health care professional. The intended environments for use include the home, hospitals and clinics.

**Rx Only**